**PRIDE**<sup>®</sup>  
MOBILITY

**Jazzy**  
ASSIST

**Quick Operation Guide**

## **Jazzy Assist Quick Operation Guide** ..... 01

Unfold  
Activate the battery (First use)  
Introduction to the Controller  
Connect to the App  
Charging Introduction  
Fold

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# Jazzy Assist Quick Operation Guide


This Quick Operation Guide is intended to provide basic instructions for using the device and to help you quickly master its operation. For safe operation, please be sure to carefully read the **Product User Manual** before use.

# Unfold

---


- 1 Step on the folding footrest, lift the handle upwards and push it forward

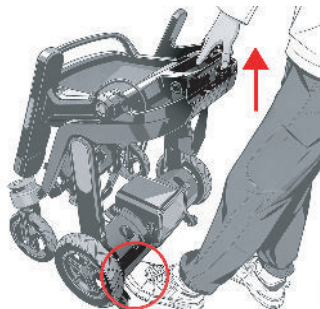
- 2 Open the backrest

- 3 Lift the armrests on both sides

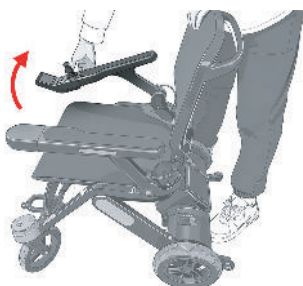- 4 Lower the footrest
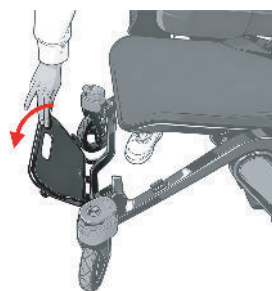
# Activate the battery (First use)


---

Press and hold the battery button for 3 seconds. The wheelchair can only be switched on after the battery has been activated.

# Introduction to the Controller

---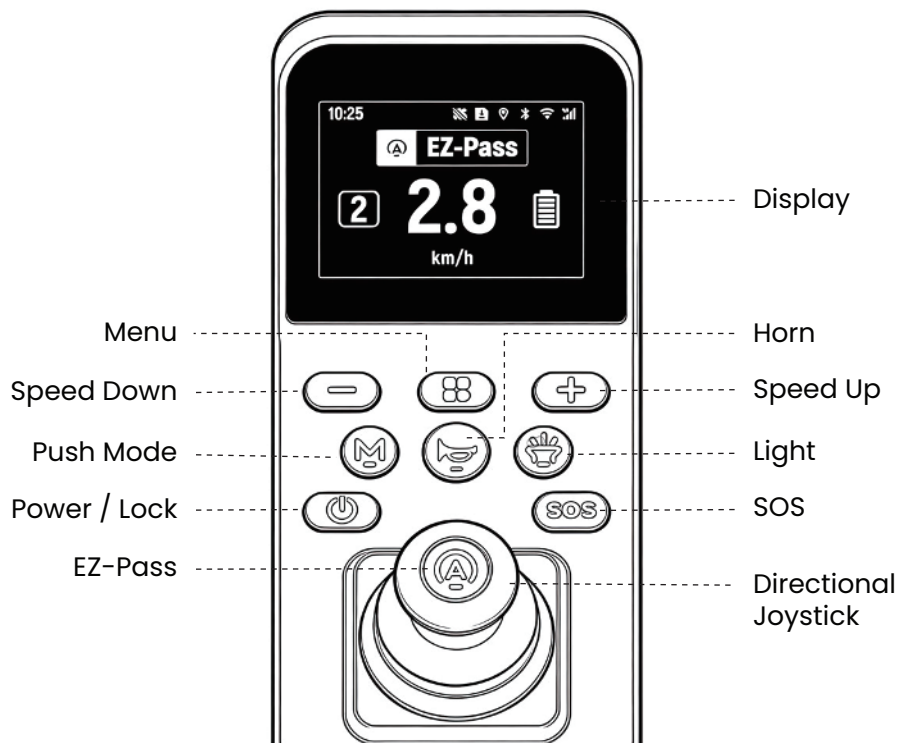

## Directional Joystick

When the wheelchair is powered on, move the joystick to control speed and direction. Pull the joystick backward to display the rear-view camera image on the screen.

## Power/Lock

Short press to power on. Press and hold for 3 seconds to power off. When powered on, short press to enter lock mode; the joystick is disabled.

## EZ-Pass

Short press to enable this mode. When enabled, it makes minor steering corrections to help Powered Wheelchair users pass through narrow doors or passages more easily. Short press again to disable this mode.

## Horn

Press to sound the horn.

## Light

Short press to turn the lights on or off.

## **SOS** SOS

Press and hold for 3 seconds to sound the alarm. Press and hold for 5 seconds to send an SMS message to the emergency contact (Only effective when the wheelchair is connected to the mobile app).

## **Menu**

Short press to enter the menu. Use the [speed up]/[speed down] buttons to switch pages. In the menu, press and hold the [speed down] button to display the exit prompt, then short press the [Menu] button to exit.

## **Speed Up**

Short press to increase by one level, up to a maximum of Level 5. In the menu, short press to go to the next page. On the volume adjustment screen, short press to increase volume.

## **Speed Down**

Short press to decrease by one level, down to Level 1. In the menu, short press to go to the previous page. On the volume adjustment screen, short press to decrease volume.

## **Push Mode**

Press and hold the [Push] button for 3 seconds to release the brakes so the powered wheelchair can be pushed manually. Short press the [Push] button or move the joystick to return to electric drive mode. You can change the button function in the app.

## **Connect to the App**

---

1. Scan the QR code to download and install the companion app for the Powered Wheelchair.

2. Open the app, then follow the in-app instructions to add the device.
3. For more information on additional features (such as SOS emergency assistance and app remote control), please refer to the app.

# Charging Introduction

## Method 1:

Connect the charger. Align the red dot on the plug with the red dot on the charging port under the armrest, and insert it in the same direction to start charging.


## Method 2:

Remove the battery. Align the red dot on the plug with the red dot on the battery charging port, and insert it in the same direction to start charging.

# Fold


1 Slightly lift the armrest, press and hold the lock release button, and lower the armrests on both sides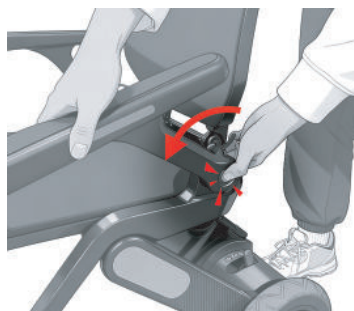

2 Step on the folding pedal, lift the backrest handle and then release your foot



3 Pull back the locking link and fold the backrest

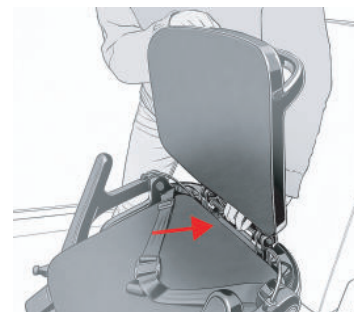4 Retract the footrest (optional)

5 Pulling the locking mechanisms on both sides of the front support wheel allows the front support wheel to be bent backward (not mandatory)

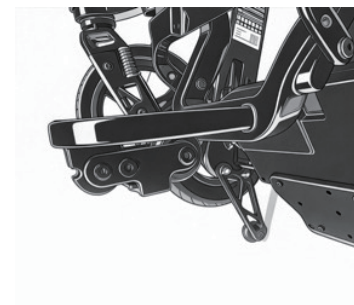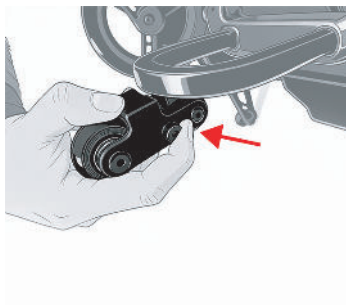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Pride Mobility Products Ltd.
Pride Mobility Products GmbH